


# ANTEATER

## INSTRUCTIONS

WWW.ANTEATERADJUST.COM

SLIDE YOUR HAND IN TO THE ANTEATER SO THAT THE 1ST KNUCKLE OF YOUR INDEX FINGER CONTACTS THE FELT PAD.

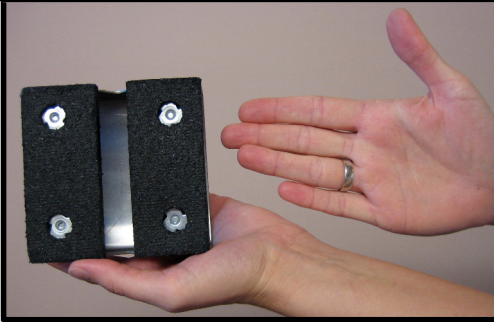

THE RUBBER PADS ARE ON THE PALM SIDE AND YOUR THUMB STICKS OUT.

USE YOUR THUMB TO PALPATE AS YOU ARE FIRST LEARNING TO USE THE ANTEATER.

EVENTUALLY YOU WON'T NEED TO.

PLACE THE ANTEATER ON THE PATIENT'S BACK WITH THE RUBBER PADS ON OPPOSITE SIDES OF THE SPINE.

USE IT ANY WAY YOU PREFER TO ADJUST!

HERE IT IS WITH THE DOCTOR USING THE OPPOSITE HAND.

ROLL THE PATIENT BACK

HERE THE DOCTOR IS REACHING AROUND THE PATIENT WITH HER OPPOSITE HAND

HERE THE DOCTOR USES HER LEG TO HELP WITH THE THRUST.

SOME DOCTORS PREFER TO ROLL THE PATIENT TO THE SIDE TO ADJUST ANTERIORS. THIS IS EASILY DONE WHILE USING THE ANTEATER.

# FIN